

calypsoyamba
clarenc coast *holiday* parks

Free beach wheelchair hire
Contact the park office for bookings
info@calypsoyambaholidaypark.com.au
Phone: 02 6646 8847



Safety Instructions




clarence
VALLEY COUNCIL

Safety Procedures

- Be aware of weather, tides, waves, wind, storms, and heat. Do not use the wheelchair in poor or adverse conditions.
- Never leave the user unattended. The wheelchair is not a flotation device. Non-swimmers should wear a life jacket.

At least two fit support people are recommended to assist the user safely on and off the beach and through sand.

Using the Beach Wheelchair

The wheelchair is designed for use at the water's edge and remains stable in water up to the centre of the rear wheels (approx. 24cm/10 inches). Its balloon tyres may float depending on the user's weight.

1. The user must not operate the chair alone. At least two support people are required.
2. Medical clearance is advised before booking, especially for users with back pain, balance issues, or pressure sores.
3. Maximum weight capacity is 120kg.
4. Never leave the chair unattended, particularly near water or slopes.

Anti-tip Assembly/ Parking Brake

Located at the rear below the handrail (Figure 1), the anti-tip bar is stored on the top crossbar when not in use.



Figure 2



Figure 3

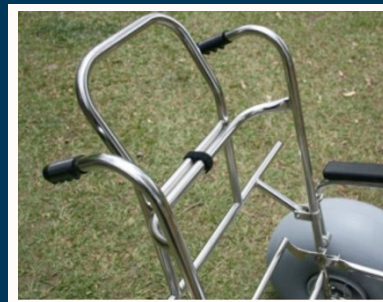


Figure 1

To apply brake:

Swing the bar down from the crossbar (Figure 2), place your foot on the middle of the bar, and push against the tyres until the locking bar engages (Figure 3). This prevents accidental release.

After Using the Beach Wheelchair

1. Return the chair in clean condition—brush off sand and wipe if needed.
2. Report any damage or maintenance issues.
3. Return the chair to Calypso reception.

To Release the Park Brake

Ensure the chair is steady. Lift the locking bar upward (Figures 4 & 5), then secure it to the top crossbar with the strap (Figure 6).



Figure 4



Figure 5



Figure 6